TRUST

BECAUSE SO MANY RELY ON YOU...
YOU CAN RELY ON US.

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Lisi PRESS:

THE NOVEL LITHIUM DISILICATE,
DO YOU KNOW ITS FACTS?

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RESTORING HEAVILY DAMAGED
MOLAR WITH COMPOSITE OVERLAY
USING INJECTABLE MOULDING
TECHNIQUE AND HIGH STRENGTH
FLOWABLE COMPOSIT

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DEEP MARGIN ELEVATION (DME) WITH INJECTABLE FLOWABLE COMPOSITE
(G-AENIAL™ UNIVERSAL INJECTABLE) AND USING FIBER-REINFORCED
COMPOSITE (EVERX FLOW™) ON CRACKED TOOTH

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ART & GRAPHIC DESIGN

Always Together Studio
& BAAN by Studio Room

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Best ClassII Corrector
Award 2023 by the American
Association of Orthodontists

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Lisi Press: The Novel Lithium
Disilicate, Do you know its facts?

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Deep margin elevation (DME)
with Injectable Flowable Composite
(G-aenial™ universal injectable)
and Using Fiber-Reinforced
Composite (everX Flow™) on
Cracked Tooth

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A MESSAGE FROM

ACCORD HENRY SCHEIN

Dear Henry Schein customers and all dental practitioners and labs in the Kingdom of Thailand.

I'm honored to write to you from my home in New York, where as CEO of Henry Schein's Strategic Business Group, I lead a global team of diverse TSMs spread across dozens of countries and businesses. New York may be my home, but my office is often an international flight from one corner of our Company to another.

This international perspective has long given me a unique view of our business, as I have learned so much about the different markets we serve, the needs of dental professionals around the world, and the cultural differences that define our team, our customers, and their patients. One of the most important things I have learned is that we all share a desire to help dental professionals deliver high-quality care for their patients.

“It's not just about technology, but the benefits that technology brings when leveraged by dental professionals like you.”

That can take many different approaches, of course, as our customers pursue a range of specialties and require a wide variety of products, services, solutions, and equipment. Fortunately, our One Schein approach to leveraging our complete portfolio of products and services helps ensure that the Thailand dental community can Rely On Us for whatever is needed — from gloves to equipment to implants to financial services and more.

The power of One Schein is demonstrated by the advancement of dental technology. Clinical workflows are increasingly digitized, and Henry Schein offers seamless, easy-to-use clinical digital workflow solutions that foster open architecture. Why? Because technology innovations are happening so quickly that a closed system quickly becomes an obsolete one. An open system offers dental professionals greater flexibility to use their preferred products without sacrificing interoperability.

In addition, Henry Schein's digital workflow solutions strengthen case acceptance and enhance the patient experience. It's not just about technology, but the benefits that technology brings when leveraged by dental professionals like you. For example, Team Schein recently supported an oral health clinic in an underserved community in the Bahamas that treated more than 1,200 people in just eight days, due in no small part to technologies like 3-D printing. Being able to create dentures and other materials on-site expands access to care in developing regions, and helps create transformative experiences for communities in need.

Our Accord team understands the power of digital technology when combined with the human touch. Together we can help you build more efficient, profitable practices that deliver high-quality care and improve oral health throughout the Kingdom of Thailand.

We thank you for your trust and your partnership, and we look forward to continuing to serve our Thai customers long into the future. Thank you.

David Brous
CEO Strategic Business Group
Henry Schein

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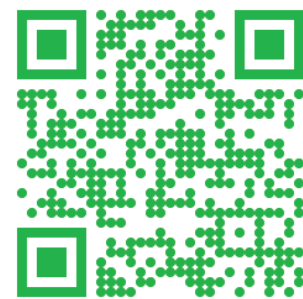
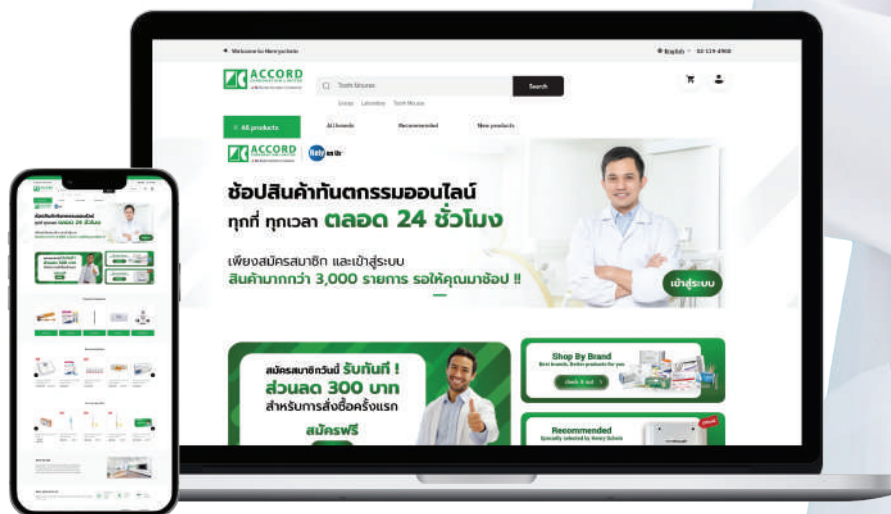
SCAN ME

www.accordhenryschein.com

ACCORD
CORPORATION LIMITED
A HENRY SCHEIN COMPANY

Rely on Us

GC

ACCORD
CORPORATION LIMITED
A HENRY SCHEIN COMPANY

Rely on Us

EQUIA Forte™ HT

Restorations that cross generations

Strong & durable
EQUIA Forte Coat nano-fillers
increase surface hardness and
wear resistance for long-lasting
restorations

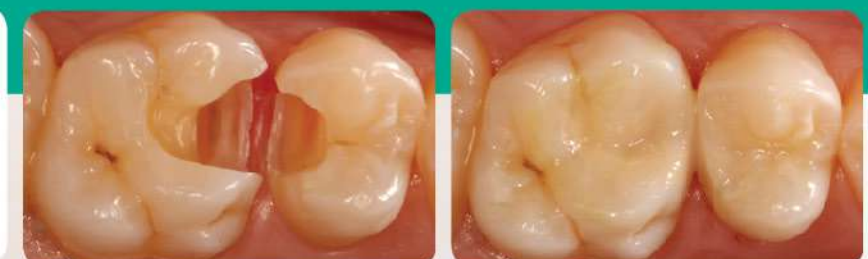
Fast & easy to handle
Packable, non-sticky and
moisture-tolerant,
fast and easy bulk placement

Increased translucency

EQUIA Forte HT is a reliable
system based on 12 years
of clinical experience with
the EQUIA family

Bulk fill glass hybrid restorative system

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DMG ICON
♦ **MINIMALLY INVASIVE DENTISTRY** ♦

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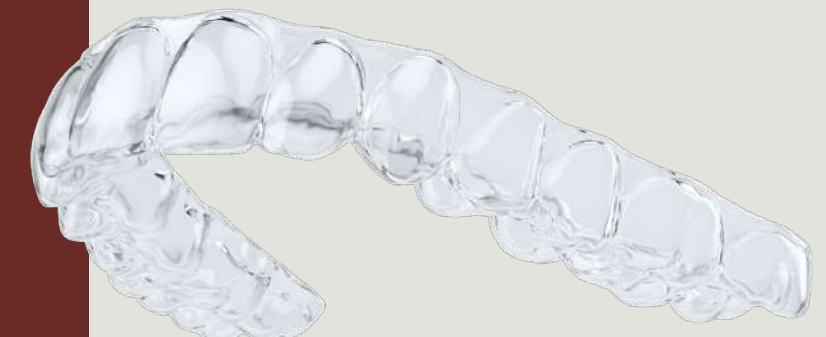
EMS AIRFLOW®
PHOPHYLLAXIS MASTER
♦ **GUIDED BIOFILM THERPY** ♦

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IVOCLAR
IPS E.MAX ZIRCAD PRIME
♦ **“ONE DISC SOLUTION”** ♦

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Once the white spots are gone the glowing smile returns.

Your work has a far-reaching influence. Apart from the medical aspects, good-looking teeth result in an added self-assurance and sense of confidence. We at DMG support you and your patients. For example, with microinvasive infiltration therapy for effective treatment of white spots.

Dental Milestones Guaranteed. Discover more from DMG at www.dmg-dental.com

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GBT (Guided Biofilm Therapy) No pain technology

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AFTER GBT

CLEAN TEETH - HEALTHY LIFE

WHY PROFESSIONAL TOOTH CLEANING IS SO IMPORTANT TODAY

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camlog®
a perfect fit

The CONELOG® Implant System

High-precision conical Connection¹⁻²
Prosthetic excellence at bone level

- **Threads down to the apex**
Ideal for immediate implantation
- **Coronal anchorage thread**
Crestal thread for improved stability in reduced bone height
- **Parallel-walled section**
flexibility of the vertical position
- **Intelligent thread design**
Deeply engaging threads
Improved distribution of axial forces
Defined bone compression
- **Anatomically shaped conical area**
Excellent primary stability
Reduced diameter for areas with anatomically challenging situations

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HENRY SCHEIN® SOLUTIONS FOR HEALTH CARE PROFESSIONALS

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EDGEFILE® X7 **PERFORMANCE** **TECHNOLOGY**

Unmatched Flexibility

Heat-treated FireWire™ NiTi gives amazing flexibility, capable of 90 curves.

NO BOUNCE BACK

FireWire™ NiTi doesn't bounce back to preserve apical anatomy

INCREDIBLE STRENGTH

Amazing strength means the confidence to tackle any case.

Compatible

EdgeFile® is fully compatible with your current system meaning a seamless transition.

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- Doctor of Dental Surgery DDS, Srinakharinwirot University
- Cert. Residency training in Prosthodontics, University of Illinois at Chicago, USA
- Clinical Fellowship in Prosthodontics and Implantology, University of Illinois at Chicago, USA
- Clinical Fellowship in Dental Oncology and Maxillofacial Prosthodontics, MD Anderson Cancer Center, Texas, USA. Diplomate, The American Board of Prosthodontics.

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			Monolithic			Bilaminate	Cutback
	Esthetic Zone	Functional Zone	Resin-Matrix Ceramics	Polycrystalline Ceramic	Glass-Matrix Ceramics	Layered Polycrystalline Ceramic	Glass-Matrix Ceramics with Incisal Cutback
Single Unit							
Anterior	✓		✓	Cubic Containing	✓	✓	✓
Premolar	✓	✓	✓	✓	✓	✓	✓
Molars		✓		✓	✓	Use with Caution or Only Layer Non-fxl Areas	
Partial Coverage	Veneers	Inlays, Onlays	✓		✓		✓
Discolored Tooth	Veneer or Crown	Crown		✓	High-Opacity Ingot	✓	High-Opacity Ingot
Fixed Partial Denture							
Anterior	✓			Cubic Containing	✓	✓	✓
Premolar 3-Unit	✓	✓		✓	Use with Caution	✓	
Molar 3-Unit		✓		✓		Use with Caution or Only Layer Non-fxl Areas	
Long span	✓	✓		✓		Use with Caution or Only Layer Non-fxl Areas	
Precision Keyways	✓	✓		✓		Use with Caution or Only Layer Non-fxl Areas	

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Reference

- Ottawa (ON): Canadian Agency for Drugs and Technologies in Health Porcelain-Fused-to-Metal Crowns versus All-ceramic Crowns: A Review of the Clinical and Cost-Effectiveness [Internet].; 2015 May 29. PMID: 26180882.
- McLaren EA, Whiteman YY. Ceramics: rationale for material selection. Compend Contin Educ Dent. 2010;31:666–668.
- Phark JH, Duarte S Jr. Microstructural considerations for novel lithium disilicate glass ceramics: A review. J Esthet Restor Dent. 2022 Jan;34(1):92-103. doi: 10.1111/jerd.12864. Epub 2022 Jan 7. PMID: 34995008.

3Shape TRIOS MOVE+

Excite patients and advance case acceptance

Join the **MOVE**ment towards next-level patient excitement

3Shape TRIOS MOVE+ allows you to easily share your digital scans, images, and treatment plans with patients. You can now excite your patients by inviting them into a transparent and open dialogue about their treatments right from their chair. This helps patients picture their final results and ultimately, advance case acceptance.

Find out more at 3shape.com/jointhemovement

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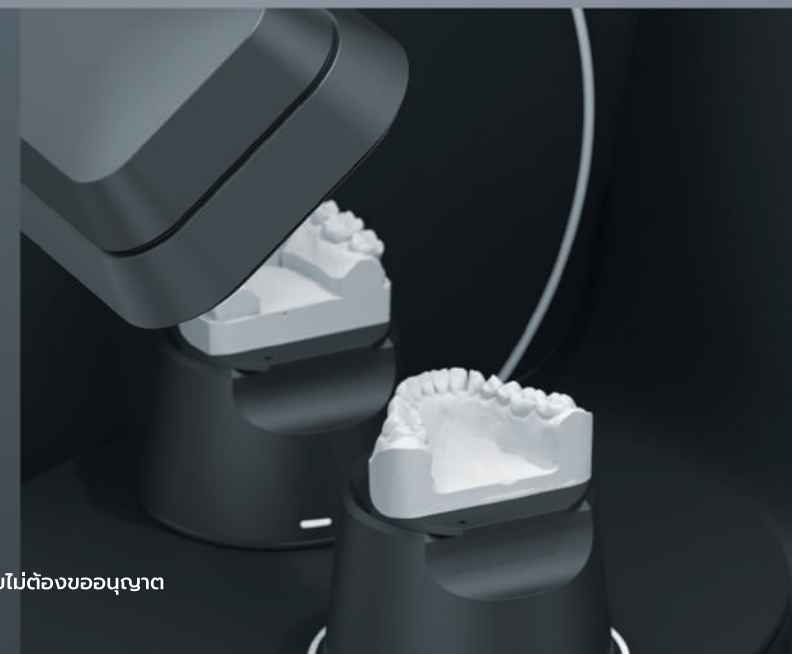
More at once

Gain the power
of dual model
scanning


Minimize hands-on time with the intuitive place-and-scan design. Provide a complete range of quality services to your clinics with ease. Stay on top of your caseload with fast automated scan-to-design workflows. More efficiency. More versatility. More speed.

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Natural beauty restored in one appointment

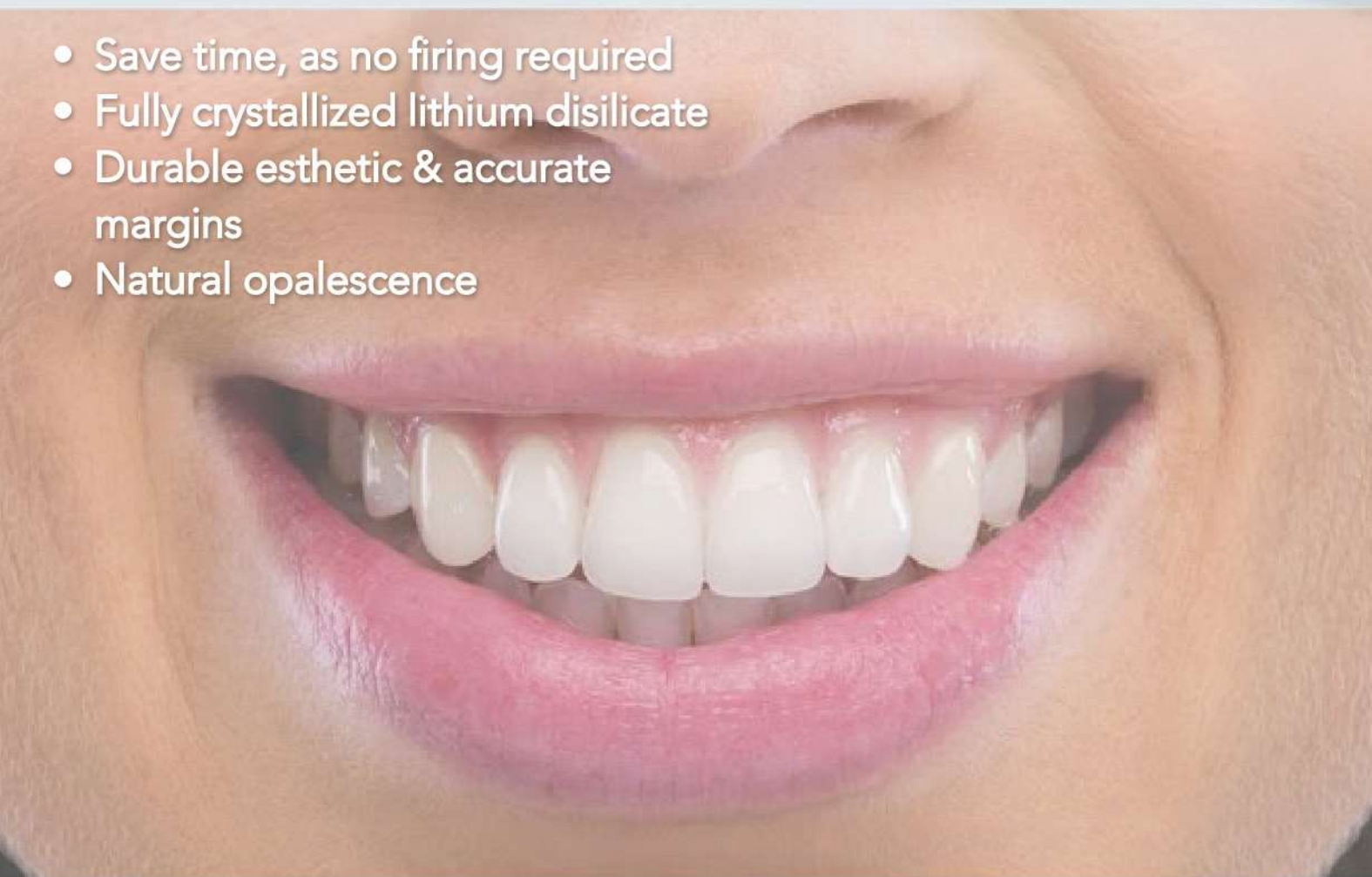- Save time, as no firing required
- Fully crystallized lithium disilicate
- Durable esthetic & accurate margins
- Natural opalescence

Just Mill, Polosh & Place

initial
LiSi Block

Lithium Disilicate
CAD/CAM Block for
chairside solutions

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HEINE ML4 LED HEADLIGHT

Perfect fit. Perfect view.
HEINE ML4 Headlight

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LiSi PRESS: THE NOVEL LITHIUM DISILICATE, DO YOU KNOW ITS FACTS?

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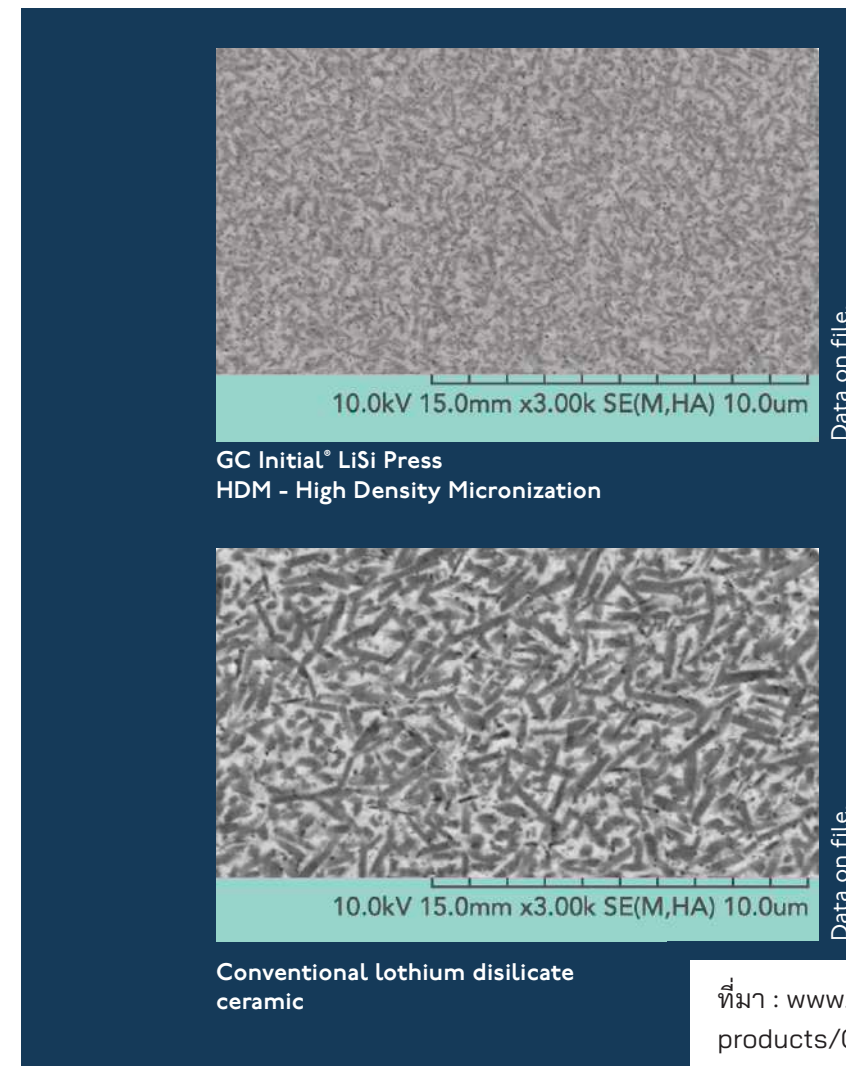
GC Initial™ Lisi Press-Medium Translucency

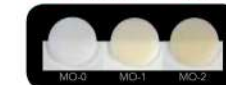
GC Initial™ Lisi Press-Low Translucency

GC Initial™ Lisi Press-Low Translucency One Body Concept

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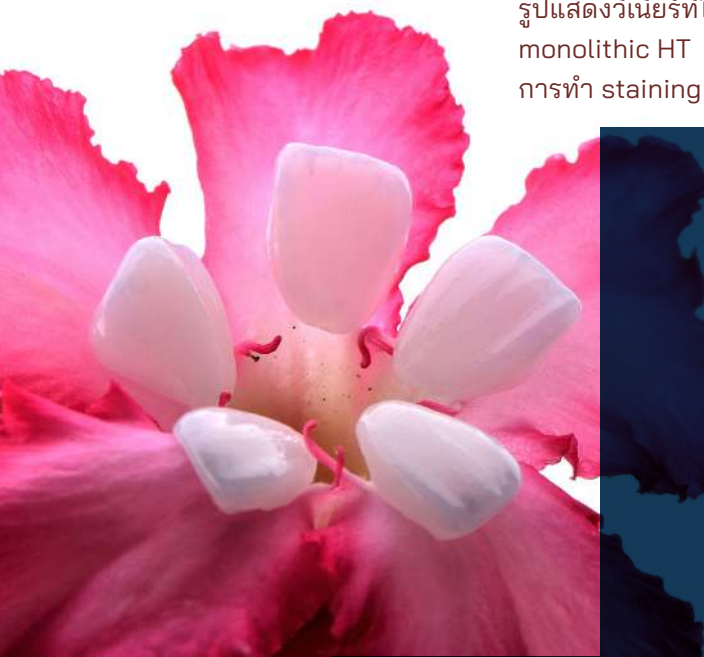
Properties (as provided by manufacturers)	Units	IPS e.max Press	Initial LiSi Press
Manufacturer	—	Ivoclar Vivadent	GC
Components	—	lithium disilicate crystals (approx. 70%), Li2Si2O5, embedded in a glassy matrix	lithium disilicate micro-crystals equally dispersed in a glass matrix
Crystal system	—	lithium disilicate - crystals measure 3 to 6 µm in length.	lithium disilicate - crystals measure 1.5 µm x 0.5 µm
Flexural Strength	MPa	433*	454*
Biaxial Flexural Strength	MPa	> 500	> 500
Vickers hardness		(HV10) 5900 ± 100 Mpa	600 HV
Chemical solubility	mg/cm2	40 ± 10	5.4 µg/cm ²
Liner thermal expansion CTE	× 10 ⁻⁶ /K	Coefficient of thermal expansion (100–400 °C) 10.15 ± 0.4 10 ⁻⁶ K ⁻¹ Coefficient of thermal expansion (100–500 °C) 10.55 ± 0.35 10 ⁻⁶ K ⁻¹	Liner thermal expansion CTE (25–500 °C) 9.8 × 10 ⁻⁶ K ⁻¹
Glass transition temperature	°C	560	520
Density	g/cm ³	2.5 ± 0.1	2.4

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Reference

- 1) Cagidiaco et al. A pilot trial on lithium disilicate partial crowns using a novel prosthodontic functional index for teeth (FIT). BMC Oral Health. 2019;19:276
- 2) Elnaggar et al. Effect of additional pressing on the color and translucency of pressable ceramic materials: An in vitro study. J Prosthet Dent. 2021;126:588.e1-588.e5

Need High-Quality Burs ? Get Microcopy!

Superior Cutting Efficiency:

- NeoDiamond burs have up to 60% diamond exposure which is 20% more than the industry standard.
- Clinically proven to cut the tooth faster and more accurately.
- Reduced chair time.

Safeguards your Handpiece:

- Unique shank design guarantees that the bur perfectly fits into the handpiece, reducing handpiece wear and eliminating 'bur chatter'.

Exceptional Strength:

- NeoDiamond burs are made from Swiss medical grade stainless steel for high durability
- NeoBurr carbide burs with a 'Blended Neck' design are 70% stronger than competitive burs.

Highest Level of Infection Control:

- Pre-sterilized burs limit the risk of cross contamination and ensure the highest level of patient safety.

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DENTAL NEEDLE

STERILE, SINGLE-USE NEEDLE FOR DENTAL PROCEDURES

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RESTORING HEAVILY DAMAGED MOLAR

with

COMPOSITE OVERLAY USING INJECTION MOULDING TECHNIQUE

and
**HIGH STRENGTH
FLOWABLE
COMPOSITE**

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- Doctor of Dental Surgery (DDS, 2nd Class Honor), Khon Kaen University
- Post graduate program in Restorative Dentistry, Chiang Mai University
- 2017- 2019: General dentist and Academic staff at Faculty of Dentistry, Chiang Mai university
- 2019 - 2022: Private practice a private dental clinic (Chiang Mai, Thailand)
- 2022 - Present: Full-time restorative dentist at Nana Dental Clinic (Hua hin, Prachuap Khiri Khan, Thailand)

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G-aenial® Universal Injectable

A Game Changer in Restorative Dentistry

Transform
the way you Work

Start injecting with our strongest direct restorative ever*

Our strongest direct restorative ever*

Dr. Javier Tapia Guadix - Spain

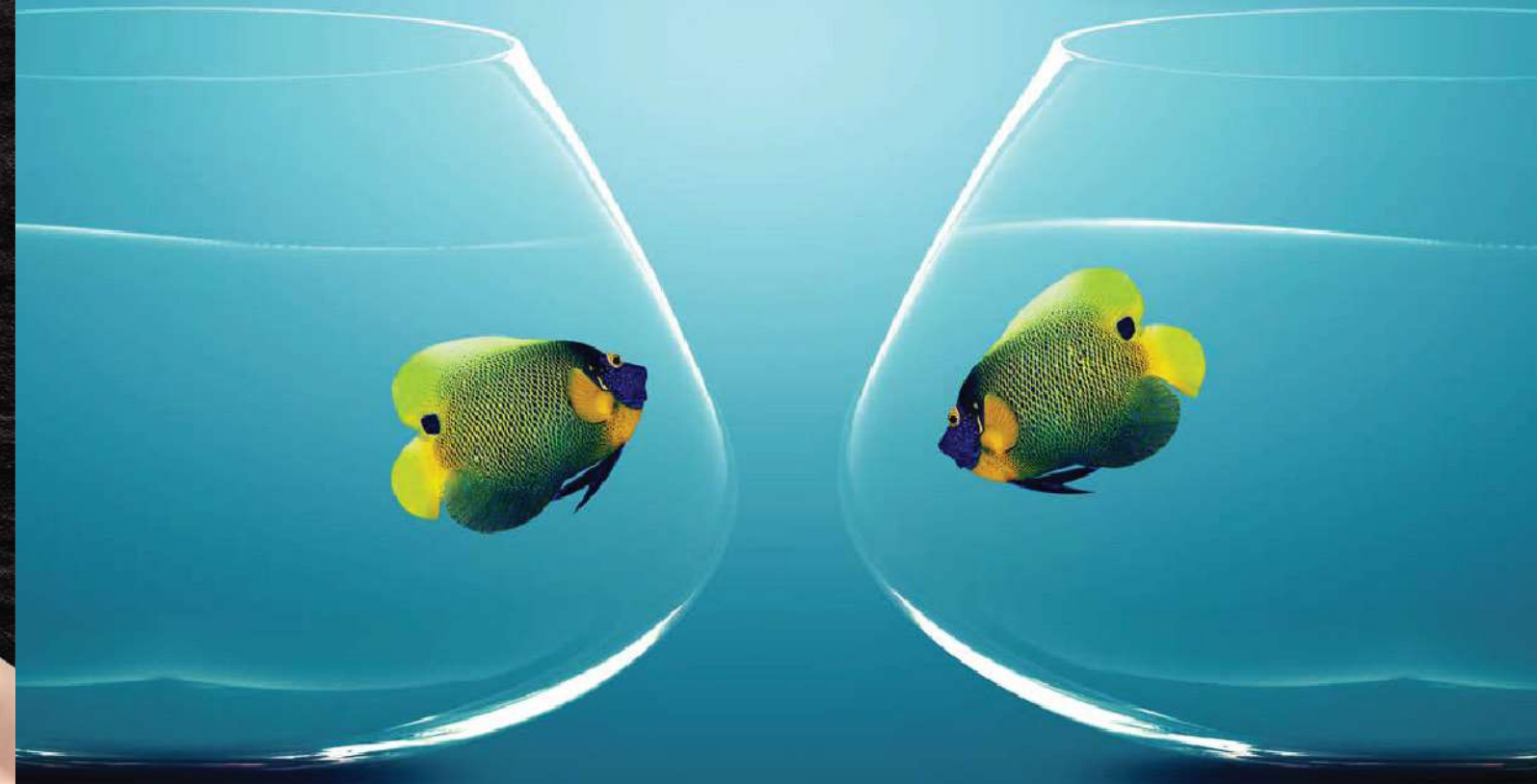
- 3 steps in 1 - Shape and contour as you inject
- Unsurpassed polishability with beautiful aesthetics
- Universal application for all cavity classes

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Seeing is believing. *Introducing*

EXACLEAR

Crystal clear vinyl polysiloxane material

Crystal clear for absolute control

EXACLEAR is an innovative, clear silicone material that fulfils the needs of demanding aesthetic cases and facilitates **CONSISTENT** results. Its absolute transparency offers total visual **CONTROL** making it ideal for **CREATING** outstanding aesthetic results in complex clinical situations.

EXACLEAR

Competitor A

Competitor B

Excellent light infiltration during injection moulding technique.

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Small Equipment & Accessories

- Light and compact curing light with a curing output of 1200mw/cm2.
- 3 curing modes: full power, ramp and pulse.
- Wave length 420-480nm.

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Safety and
protection

Absolute dryness

Unblocked field
of vision on the
treatment area

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DEEP MARGIN ELEVATION (DME) WITH

INJECTABLE FLOWABLE COMPOSITE (G-AENIAL™ UNIVERSAL INJECTABLE)

AND USING FIBER-REINFORCED COMPOSITE (EVERX FLOW™) ON CRACKED TOOTH

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- Private Clinic Dentist at NP Dental Clinic
- Speaker at the Academy of Veneers
- Co-Founder Facebook Page BDA: Biomimetic Dentistry Academy
- Course Director of Amazing Bonded Ceramics from BDA
- Dentist and Teacher Assistant at the Faculty of Dentistry, Srinakharinwirot University

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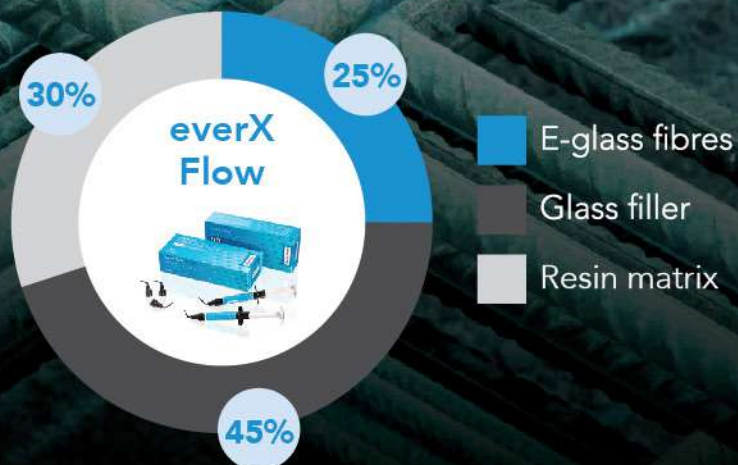
Strong to the core

everX Flow


Short-fibre reinforced flowable composite for dentine replacement


Stops fracture propagation

Crack propagation is arrested and redirected by the fibre-reinforced substructure.

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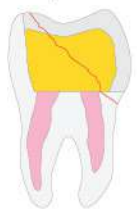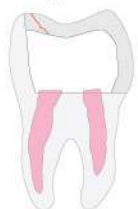Two shade options for greater flexibility

Translucent shade

Optimal for deep posterior cavities
Depth of cure: 5.5mm

Dentin shade

Optimal for more aesthetic results and for core build-up
Depth of cure: 2.0mm

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G-ænial – A universal composite that captures the genius of nature with one shade simplicity

- Beautiful natural high gloss restorations
- Simple shade selection
- Strong & Flexible
- Low stress simplicity
- Low shrinkage stress
- Firm handling

Composition of G-ænial

Improved Prepolymerized Filler

Prepolymerized Filler

FAL-silicate glass

Nano silica

G-ænial – A universal composite that acknowledges that anterior and posterior teeth are different

G-ænialANTERIOR features enhanced light scattering abilities, to bring a more natural vitality and exceptional shade matching for superior anterior aesthetics.

They have subtle optical differences...

Prof. M Peumans, Belgium

G-ænialPOSTERIOR introduces shades with a deeper concentration of colour, to better match the optical properties of posterior teeth.

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PROF. DR.CHIDCHANOK
LEETHANAKUL, Dean Faculty of
Dentistry, Prince of Songkla University

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Business models

- Strategy 2023-2025: Identity and strengthening
- Strategy 2025-2027: Growth with JCI & workshop
- Strategy 2027-2030: Improving QS ranking

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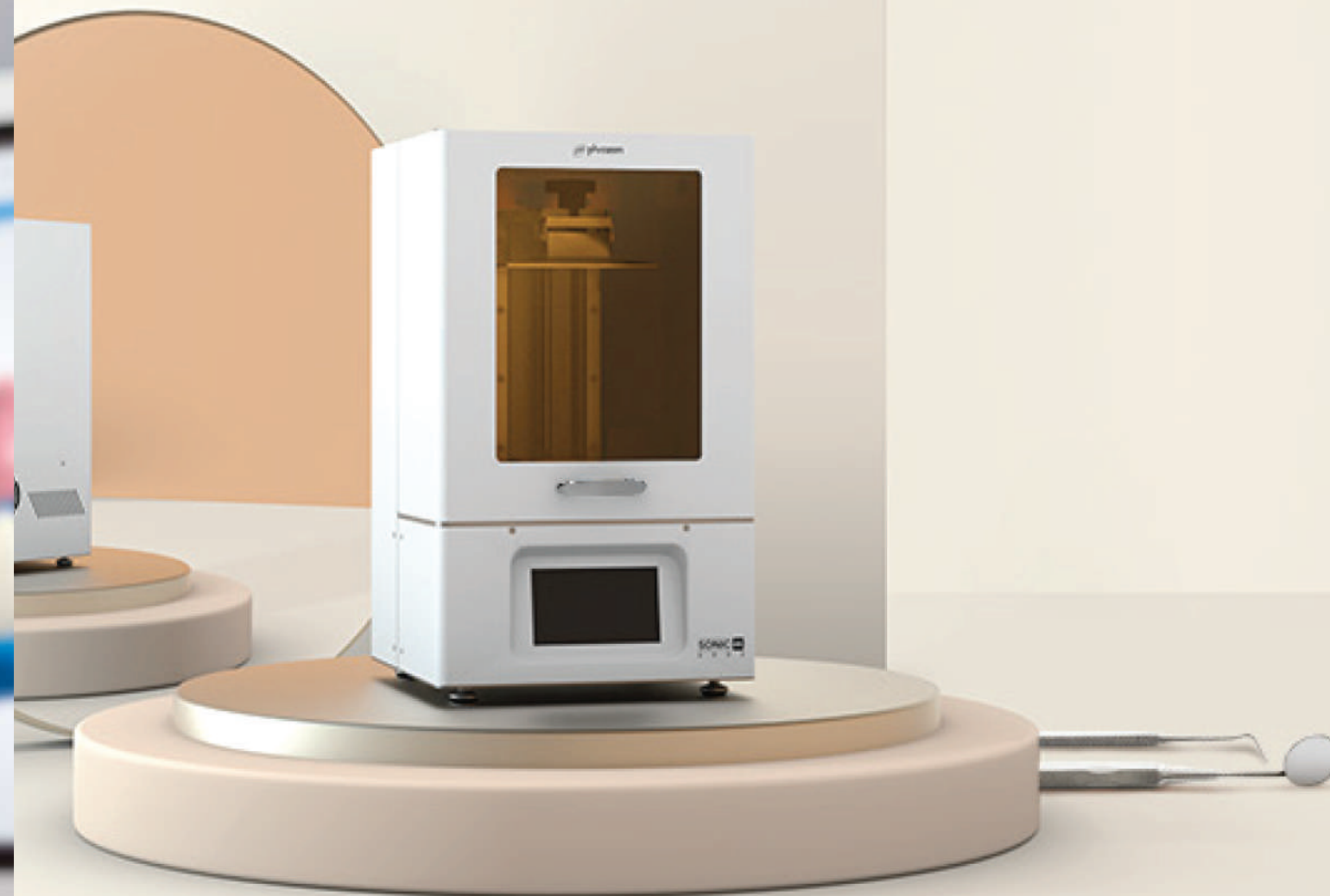
**Sonic 4K
2022**

Create Digital Models & 3D
Prints with Extreme Efficiency

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UNLIMITED DIAGNOSTIC POTENTIAL

POWERFUL DIAGNOSIS

Precise, perfectly defined FOVs ranging from 4 x 4 to 18 x 18 cm. Advanced ApT technology and teleradiographic function for high contrast, finely detailed images.

NEWTOM HR - DC TECHNOLOGY

Ongoing research and innovation have produced this device made up of innovative hardware technology and exclusive reconstruction algorithms.

PATIENT SAFETY

Low-dose protocols, SafeBeam technology and servo-assisted alignment to protect patient health.

10" TOUCH CONSOLE

NNT, the powerful imaging software with specialist interfaces and tools, user-friendly workflow and online assistance.

Perfecting your art.

ZEISS EXTARO 300

// INNOVATION
MADE BY ZEISS

EXTARO® 300 from ZEISS provides breakthrough visualization modes enabling new dental applications. It is poised to revolutionize and differentiate your practice with

- Augmented Visualization
- Digital Patient Communication
- Single-Handed Operation

Learn more about a new dimension in visualization today!

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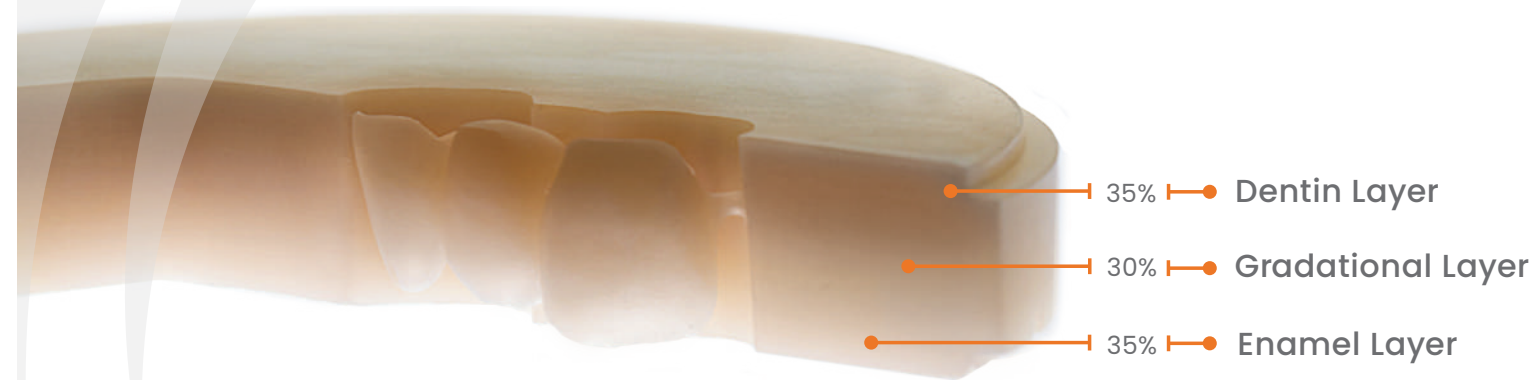
A5.

think.


- Clear information and user-friendly functions
- High performance instruments
- Integrated clinical performance
- Coordinated comfort
- Safety in the surgery
- Real-time diagnosis

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
Multi-Layered Zirconia
with more indications and strength
than Lithium Disilicate.

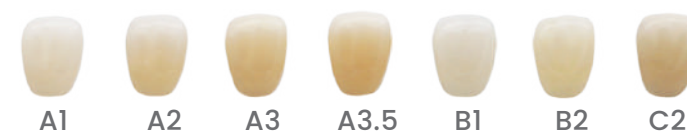Available in 98.5mm shaded discs.

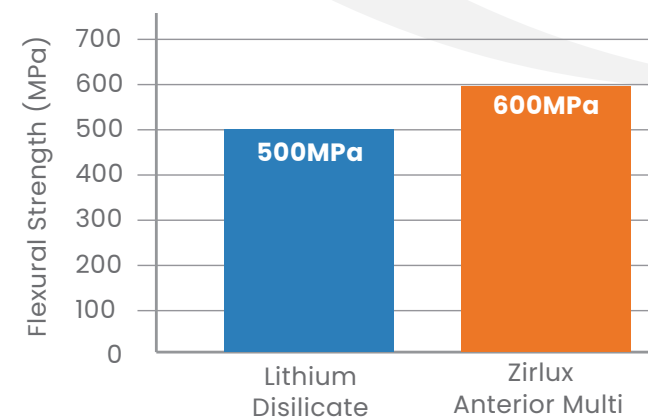Natural transition of layers mimic Incisal, Dentin, and Gingiva.

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
REMEMBERING THE LATE MR. MAKOTO YAMAMOTO

We bid farewell to a great dental ceramist, a scientist, a researcher, an author, a teacher, an advisor and a legend in the field of dental technology.

Mr. Makoto Yamamoto passed away on February 2, 2023, this is a great loss to dental technology, prosthetic dentistry and the dental industry both in Japan and around the world. We express our heartfelt condolences to his beloved family.

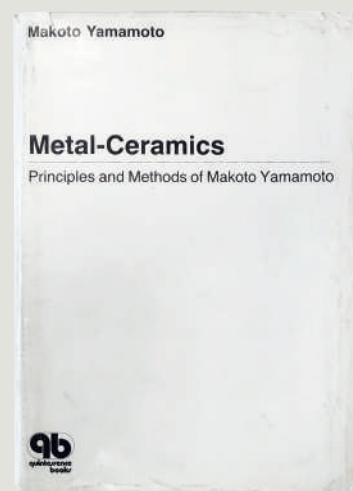He had contributed immensely towards the advancements in dental technology by developing innovative products to enhance capabilities of dental ceramics. He was honoured by Harvard University for his contribution in the field of prosthetic dentistry.

He has authored many books, including “The Metal Ceramics” a must-read for all dental ceramists and was the first dental technician in the world to receive the Lifetime Achievement Award at the 4th Japan International Dental Congress.

TRIBUTE FROM MS. SUCHADA CHARNSETHIKUL, FOUNDER AND SHAREHOLDER OF ACCORD CORPORATION LTD...

“In 1986, Accord Corporation Ltd was honored to be appointed as the exclusive distributor for the complete range of products from **SHOFU INC.**, including the lab products with the recommendation from Dr. Arob Wattanavijarn, owner of SAdental Lab and pioneer in producing PFM fabrications in Thailand.

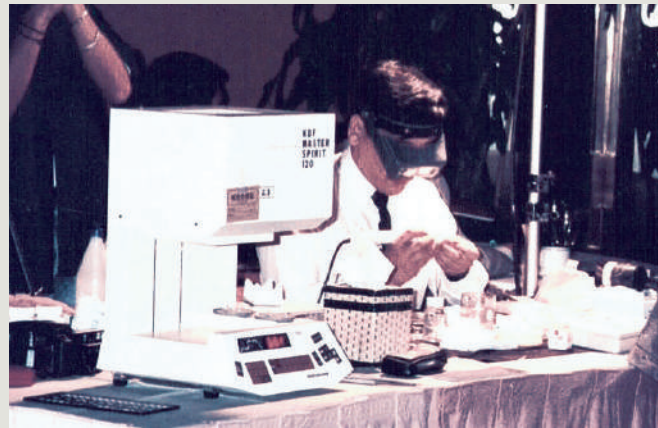
1949 - 2023

This can be considered as a monumental time where fabrication of dental prosthetics have changed in the field of dental laboratories.

At that time, **SHOFU INC.**, in collaboration with Accord Corporation Ltd. invited Master Makoto Yamamoto, Japan's most famous dental technician, to demonstrate and teach the complete PFM fabrication process including Dental Anatomy, Functional Aesthetics and Tooth Color matching with product selection, as well as techniques to create PFM restorations that look naturally beautiful.

The academic seminars and live demonstrations by Mr. Yamamoto had attracted the attention of many owners of laboratories and dental technicians including professors from the Department of Prosthodontics and Specialists in Thailand. Till to date, there are many dental lab owners and technicians who remember the name of Mr. Yamamoto with admiration.

Mr. Yamamoto, was a great educator, a researcher, a scientist, and the author of many textbooks that have been translated and taught in several countries, including "The Metal Ceramics" (referred as the bible in PFM fabrications) a book which many of our Thai dental technicians and academicians have read.

Mr. Yamamoto shared his knowledge with passion and left a lasting impression on me and many dental technicians in Thailand. He generously gave me the opportunity to accompany the lab owners and lab technicians to visit his lab in Osaka, Japan and provided a guided tour of his lab and shared expertise on managing his lab. It was an unforgettable moment and memory I cherished.

I would like to thank and pay my respects to Mr. Yamamoto, a person who is important to me, Accord Corporation Ltd. and Thai dental technicians. May his soul rest in peace."

Tribute from Mr. Patrick Loke, Executive Chairman of SHOFU Dental Asia-Pacific Pte Ltd ...

Mr. Yamamoto was instrumental in the development of a wide range of Shofu laboratory products, such as UNIBOND & VINTAGE Opal porcelain (world's first Opalescent porcelain) and other dental ceramic materials, such as the recent Vintage Art Universal Stain and Yamamoto Liquid (a color revealing aid a must have for every modern dental ceramist). He was also involved in the development of the world's first dental colorimeter SHOFU ShadeEye, and the EyeSpecial series, etc., and was active in the field of dental technology education worldwide.

Our entire SHOFU organization would like to express our eternal gratitude to Mr. Yamamoto for his contributions to the dental community and our company where his legacy will live on. He will be greatly missed not only by those who have worked with him, but also many who have known him for all these years."

"Mr. Makoto Yamamoto was SHOFU's Principal Technical Advisor and we are deeply saddened by the news of his passing and express our heartfelt condolences and deepest sympathies to his beloved family.

I was honored to be able work with Mr. Yamamoto in the late 80's. He was a perfectionist and never settled for just ordinary.

When he conducted a lecture or demonstration he always ensured that all the logistics must be very well organized and prepared in order give the best experience for the audience.

Mr. Yamamoto had taught many dental technicians and dentists throughout the world in Metal Ceramic technology and techniques, where he had left behind many imprints of his remarkable skills, he is truly a Master Dental Ceramics.

Mr. Yamamoto shared his moto with all the dental technicians and dental ceramists ...

"IN ORDER TO BE SKILLFUL IN YOUR ART OF DENTAL TECHNOLOGY, YOU MUST PUT YOUR SOUL INTO YOUR SKILLS."

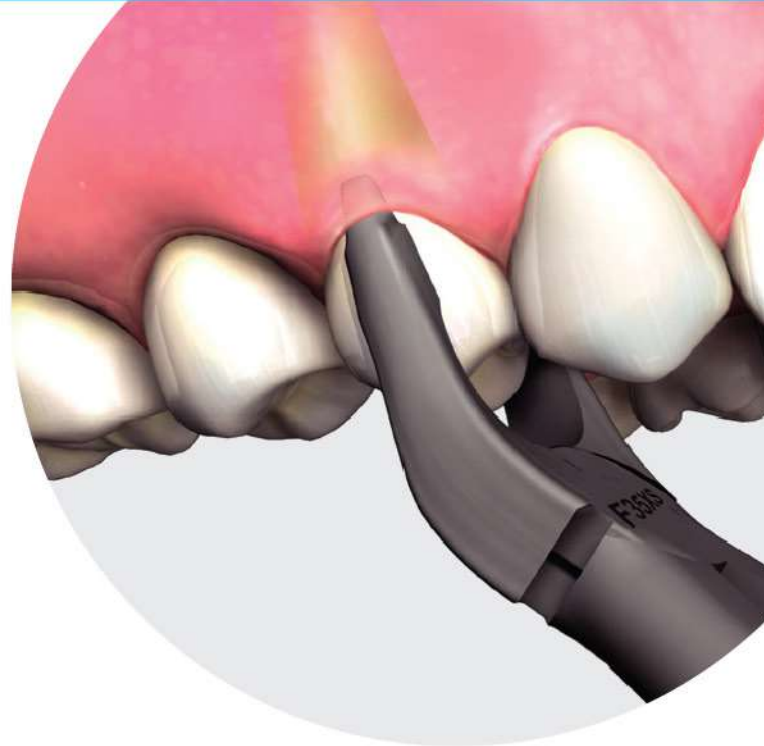
“技魂..”
Dec. 4th. 1997
Makoto Yamamoto

Introducing HuFriedyGroup

ATRAUMAIR FORCEPS

HuFriedyGroup line of Atraumair Forceps are designed for optimal access and enhanced crown and root engagement.

All forceps are meticulously crafted to the highest quality standard, ensuring superior performance and longevity. Atraumair Forceps are available in 15 unique patterns that provide a complete line for a variety of extraction cases.

ATRAUMAIR POINTS OF PERFORMANCE:

- Enhanced beak geometries and longitudinal beak serrations provide a firm, **secure grasp of the crown and root** reducing the risk of crushing or fracturing the tooth
- Laser marked hinge for **greater visibility** when bagged for sterilization
- Matte finish for **enhanced contrast and reduced light reflection**
- Unique hole design reduces weight of the forceps, providing **increased comfort and control**
- Meticulously crafted from the finest surgical grade stainless steel

HuFriedyGroup
The Best In Practice

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Smart Solutions for
Challenging Restorations

Aesthetics Powered by

Rely on Us™

Rely on Us™

Livera *α*

Alpha-CH Acrylic Teeth

For more information on **Livera *α***, please contact

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Your NEW Septodont retraction cord

100% cotton
knitted cord for
better resiliency

Septofil N

Knitted retraction cord
Non-impregnated

Features	Advantages	Benefits
100% cotton	Natural material	Makes moisture control effective while minimizing gingival irritation
Knitted retraction cord	Special structure	Better absorbency and expansion
No fluorescent whitening agent	Naturally processed cotton	Minimizes risks of side effect

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BA Optima

Cordless Endodontic Unit

BA Optima E+ Cordless Endodontic Unit

BAE370R

- 7 memory programs (5 rotating, 2 reciprocating).
- Auto-reverse and auto-stop functions.
- Speed: 100-600 rpm.
- Torque: 1-4N.cm.
- Autoclavable (head).

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Heron™ IOS
An Intraoral Color Impression
Scanner Solution

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Copyright© 2022, All rights reserved.